HUMANTECHNIK

DS-1

D

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Seite 2

Digitalwecker DS-1

GB

Operating Instructions

Page 8

Alarm clock DS-1

F

Mode d'emploi

Page 14

Réveil digital DS-1

NL

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Pagina 20

Digitale wekker DS-1

I

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Pagina 26



Sveglia digitale DS-1

E

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Página 32

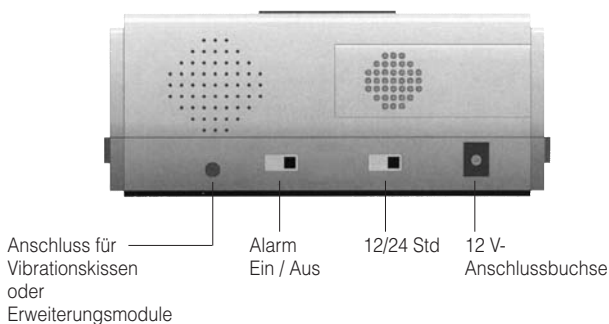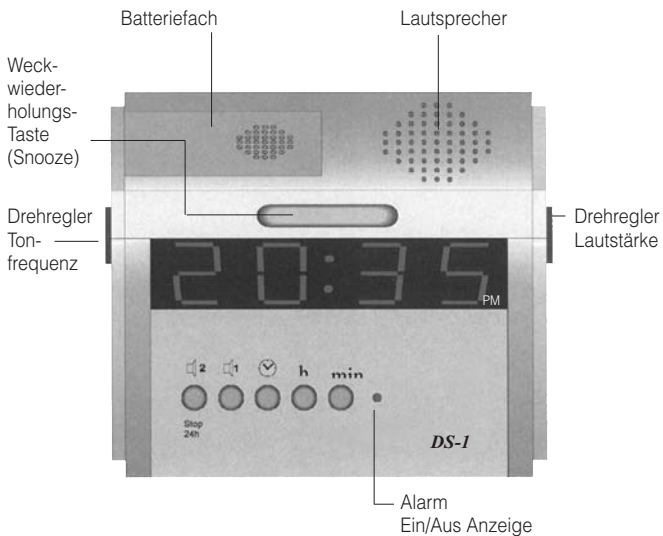Despertador digital DS-1

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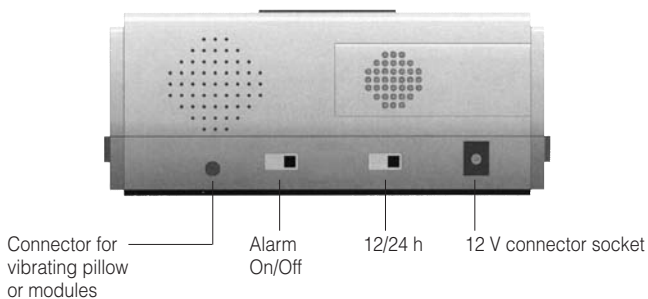
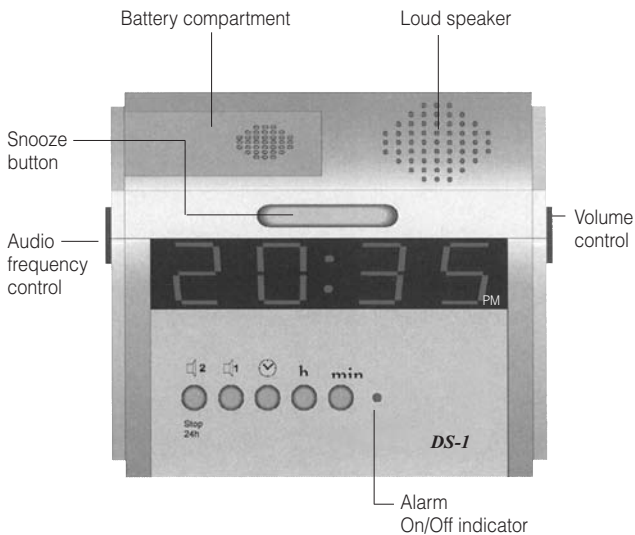
Congratulations on purchasing your alarm clock *DS-1*. We hope it will provide you with many years of service. You have chosen a modern and reliable system.

Please read through the operating instructions carefully to be able to start the unit correctly and to familiarise yourself with all of the systems features.

Standard components

Please check if all following components are included:

- *DS-1*
- 12 V power supply unit
- Operating instructions
- Warranty card

If any parts are missing please immediately contact your dealer or the manufacturer directly.

Functional principle

The alarm clock *DS-1* is an electronic clock with a digital time display.

It wakes you up with an acoustic signal which is adjustable in audio frequency and volume. The following accessories can be connected to the alarm clock:

- Vibrating pillow
- Flash module
- Switch module
- Acoustic module

Installation

Connect the power supply unit to the 12 V connector socket of the alarm clock and plug it into a 230 V power outlet. The display flashes as a sign that the time has not yet been set. The flashing disappears as soon as the time is set.

Time setting

Press  and **h** or **min** simultaneously to set the time.

Note

The time can be displayed in a 12 or 24 hour modus. The setting can be changed at the 12/24 h switch at the back of the alarm clock.

Setting the alarm time

The alarm clock has two alarm times that can be set separately.

Alarm time 1

Press  **1** and **h** or **min** simultaneously to set the desired alarm time (Duration of alarm signal 3 min).

Alarm time 2

Press  **2/Stop 24 h** and **h** or **min** simultaneously to set the desired alarm time. (Duration of alarm signal 3 min).

Note

After the installation only alarm time 1 is active.

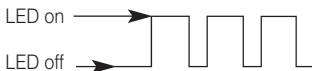
Both alarm times can be activated separately. Press **1** and **2/Stop 24 h** simultaneously and push the snooze button. With every push of the snooze button one of the three possible settings is shown on the display.

You can choose between the following settings:

Alarm time 1 active:
Display indication: AL 1
Alarm time 2 active:
Display indication: AL 2
Alarm time 1 and 2 active:
Display indication: AL 1 2

In addition to the display indication the LED flashes as follows:

Alarm time 1 active:

Alarm time 2 active:

Alarm time 1 and alarm time 2 active:
LED lights continuously

Switching the alarm on/off

Activate/deactivate the alarm with the alarm on/off switch at the back of the *DS-1*. If the switch is in the OFF-position, both alarm times are deactivated.

Setting of the audio frequency

The audio frequency of the alarm signal can be adjusted with the audio frequency control on the left side of the alarm clock. The audio frequency increases if the control is turned to the left and decreases if the control is turned to the right.

Note

The alarm tone is turned off if the audio frequency control is turned completely to the left.

Setting of the volume

The volume of the alarm signal can be changed with the volume control on the right side of the alarm clock. The volume decreases if the control is turned to the left, the volume increases if the control is turned to the right.

Snooze function

Pressing the snooze button interrupts the alarm. The alarm starts again after approx. 5 minutes.

Special features

- The alarm signal is repeated after 4 minutes (auto snooze) unless you press the snooze button or the Stop 24 h button.
- If a 9 V battery is inserted into the battery compartment the alarm clock operates even in case of a power failure. The display of the alarm clock turns off but the alarm signals are triggered.
- The inserted 9 V battery can be checked by pressing the snooze button. If "LO" is indicated on the display of the alarm clock, the battery has to be changed as otherwise the power failure back-up cannot be guaranteed.

Maintenance and care

The *DS-1* does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents.

Do not set up the unit where it will be exposed to full sunlight for long periods. In addition it must be protected against excessive heat, moisture and severe mechanical shocks.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Please make sure that the batteries are not exposed to excessive heat from such as sunlight, fire or anything similar.

Warranty

The *DS-1* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer. This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people who are not authorised to do so (destruction of the seal on the unit). Repairs under warranty are only carried out providing you have filled in and returned the enclosed warranty card from the dealer.

Always specify the product number in any event.

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

Technical data

Power supply:	12 V power supply unit at 230 V, 50 Hz
Power failure operation:	9 V battery
Power consumption:	approx. 2.5 Watt

Height:	65 mm
Width:	152 mm
Depth:	134 mm
Weight:	375 g

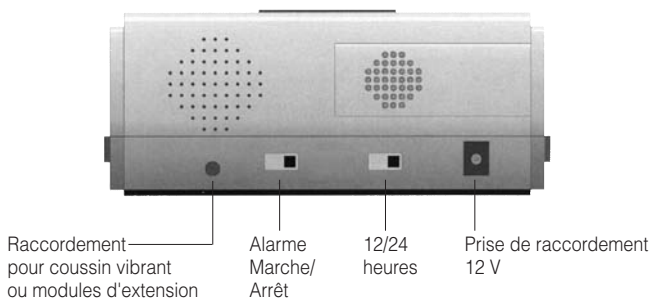
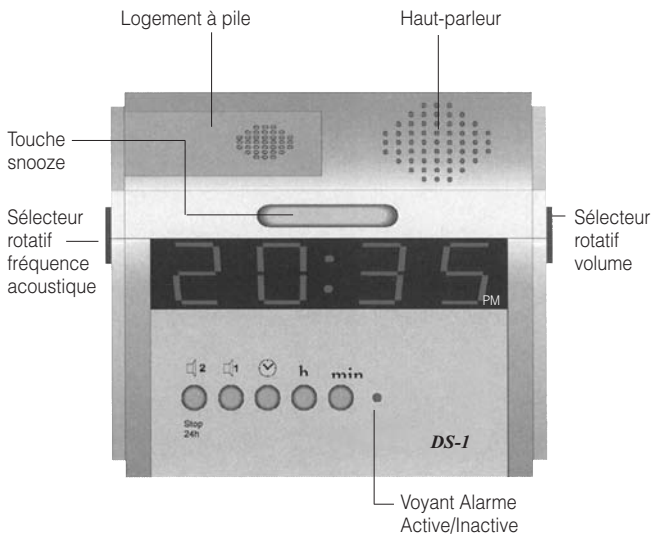
This device satisfies the following EU directives:

- 2002/95/EG RoHS directive
- 2004/108/EG EMC directive
- 2002/96/EG WEEE directive
- 2006/95/EG Low voltage directive

Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at www.humantechnik.com.

Technical specifications subject to change without notice..

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Sarabec Ltd